Sentry⁺ **Sentry⁺ HD**

User manual
Bedienungsanleitung

Optelec

Sentry⁺ Sentry⁺ HD User manual

**Version 5.0
Copyright 2013 Optelec, The Netherlands**

All rights reserved

Optelec
P.O. Box 399
2990 AJ Barendrecht
The Netherlands
Telefon: +31 88 6783-444. Fax +31 88 6783-400
E-Mail: info@optelec.nl
Internet: www.optelec.com

WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on 13th February 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will allow us to help conserving natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure safety of human health and environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

Sentry⁺ Sentry⁺ HD User manual

April, 2013

Optelec was founded in 1975 in Rockanje, the Netherlands. Since then, Optelec has developed and produced innovative products to assist blind and visually impaired people in their daily activities. Visit the Optelec website at www.optelec.com for extensive information about Optelec products and services.

All rights, in particular the rights of reproduction as well as translation, are reserved. No parts of this manual may be copied, in any form whatsoever, without written approval by Optelec.

Every effort has been made to ensure that the information contained in this user manual is correct at the time of issue. Neither Optelec, nor its agents, assume any responsibility for errors or damages resulting from use other than described in this manual.

Index

1.	Introduction	7
	About this manual	7
2.	Important safety information.....	8
3.	What's in the box?.....	9
4.	Getting to know your Sentry ⁺ [HD]	10
	Setting up your Sentry ⁺ [HD]	10
	Positioning the camera and the camera arm	12
	Close-up and distance viewing	12
	Use of the built-in object lighting	12
5.	Operation of the Sentry ⁺ [HD] control panel.....	13
	Switch On/Off.....	13
	Setting the magnification.....	13
	Wide screen mode	13
	The overview mode.....	14
	Viewing modes.....	14
	Semi-colours	14
	Auto-focus On/Off	14
	Contrast	14
	Brightness.....	15
	Activating line markers/window blinds	15
	Reading line and window shades activating	15
	Reading line colours	15
	Key switch.....	15
	Picture switch.....	15
	Reset.....	15
6.	Sentry ⁺ Specifications	16
7.	Sentry ⁺ HD Specifications	17
8.	Compliances	18
9.	Warranty Conditions.....	20
	Contact information.....	21

1. Introduction

Congratulations on choosing the Sentry⁺ or the Sentry⁺ HD, a high quality electronic magnifier designed by Optelec. Sentry⁺ [HD] is the term which is basically used for both devices in this User manual. The Sentry⁺ [HD] is an auto-focus electronic magnifier designed for use at different locations: at home, in school or at the office. Use the Sentry⁺ for reading, writing, viewing photographs, craftwork and much more.

The Sentry⁺ [HD] is designed for optimal efficiency when working with a stand-alone monitor. The Sentry⁺ [HD] can be operated ergonomically from the control panel and when used in combination with the Optelec TwinView monitor it can be connected to a computer.

If you have any questions or suggestions concerning the use of this product, please contact your Optelec distributor or the Optelec headquarters using the contact information found at the end of this user manual. Your feedback is greatly appreciated. We hope you enjoy working with your Sentry⁺ [HD].

About this manual

This user manual provides a comprehensive guide to the features and operation of the Sentry⁺ [HD] electronic magnifier. Please read this user manual carefully before using the Sentry⁺ [HD] and keep it in a safe place for future reference.

2. Important safety information

- Optelec does not assume any liability or responsibility for incorrectly detected content of texts or their interpretation by Sentry⁺ [HD]. Please have important documents checked additionally.
- Only use the Sentry⁺ [HD] indoors.
- Do not expose the Sentry⁺ [HD] to extreme or rapid changes in temperature or humidity. These conditions may damage the electronics or optics of the Sentry⁺ [HD].
- Do not place the unit on an unstable surface. It may fall, causing serious damage to the unit or injuries to the user.
- To avoid injuries, always assemble, disassemble and handle the components of the Sentry⁺ [HD] with good care as there are movable and rotatable parts.
- Do not expose the Sentry⁺ [HD] to excessive heat or direct sunlight to avoid the risk of fire.
- Do not remove any parts of the Sentry⁺ [HD]. Contact your Optelec distributor to service the Sentry⁺ [HD] if necessary. Do not open any of the components as this will void the warranty. Each component contains parts which should only be serviced by an Optelec repair specialist.
- To avoid risk of electrical damage, keep the Sentry⁺ [HD] away from fluids and chemicals
- Handle the Sentry⁺ [HD] with care. Rough handling will damage the internal components.
- Do not use the Sentry⁺ [HD] near inadequately shielded medical devices.
- Always unplug the Sentry⁺ [HD] before cleaning. Use a soft damp cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage the unit.
- Make sure you connect the Sentry⁺ [HD] to an earthed mains power (AC) outlet for your safety.
- Do not use the Sentry⁺ [HD] unit under 0° or above 40° Celsius.

Use of the Sentry⁺ [HD] other than that described in this user manual will exclude it from warranty conditions.

3. What's in the box?

The Sentry⁺ [HD] package contains the following:

- The Sentry⁺ [HD] camera arm
- Base plate for the camera arm
- Bolt and screw for assembling the base plate and the camera arm
- Control panel
- 2 different lens attachments
- Power supply
- 2 DVI-cables
- User manual

If any of these items are missing from your package, please contact your local Optelec distributor.


4. Getting to know your Sentry⁺ [HD]

Setting up your Sentry⁺ [HD]

To set up the Sentry⁺ [HD] please follow the steps described below:

1. Assemble the base plate and the camera arm by using the bolt and screw provided.
2. Place the assembled unit and the control panel nearby your monitor on a steady surface.
3. Connect the cable of the Sentry⁺ [HD] camera arm to the camera connection at the rear of the control panel (look at the picture on page 11).

4. Connect the DVI-cable of your monitor to the as “monitor” labelled connection of the control panel.
5. By using the delivered DVI-cable connect your computer to the as “DVI In” labelled connection of the control panel.
6. Additionally you can use a footswitch. Connect this to the first left connector at the rear of the control panel.
7. Connect the supplied power supply to the power connection at the back of the control box and plug the power cord into a power outlet. Your Sentry⁺ [HD] is now ready for use.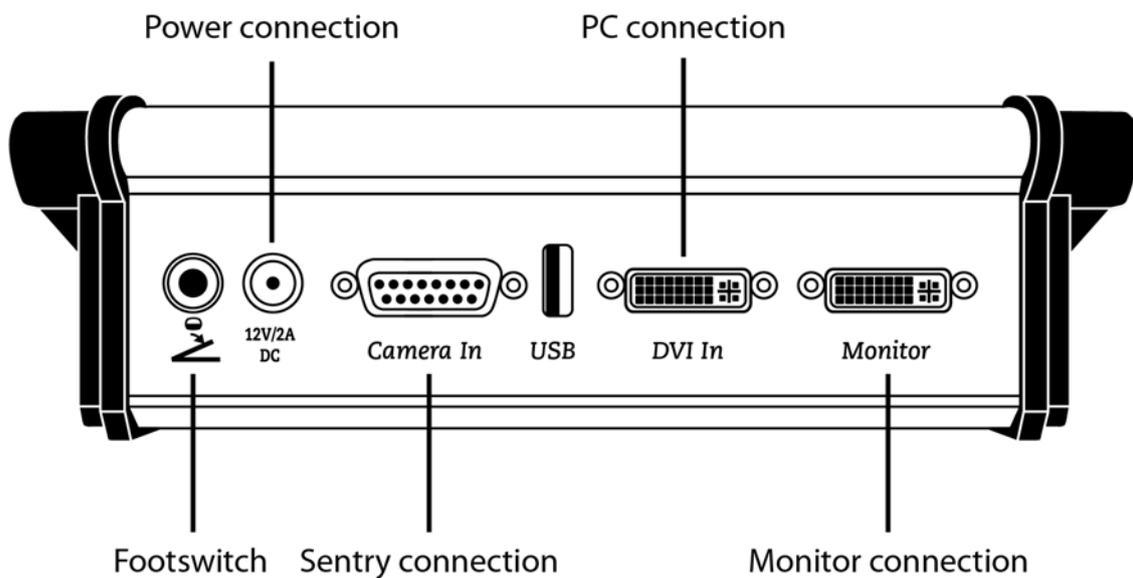


Positioning the camera and the camera arm

The Sentry⁺ [HD] features a flexible camera head and a rotatable camera arm. The camera head can be swivelled around a horizontal and a vertical axis. The camera arm can be rotated around the base. Point the camera to read a document nearby. When you want to look in the distance (for example at a blackboard) or at yourself, you need to remove the close-up lens and turn the camera in a horizontal position. By loosening the screw lock at the end of the camera arm, you can adjust the height of the camera (this will affect the magnification range).

Close-up and distance viewing

The position of the camera can be adjusted for desktop or distance viewing. To view objects within 55 cm, position the close-up lens in front of the camera unit. To view objects at a more distant proximity, position the close-up lens away from the camera. To prevent losing the close-up lens, attach the lens to the top side of the camera head.

Use the near lens with the flattened edgings for the position below and use the round lens for the upper position.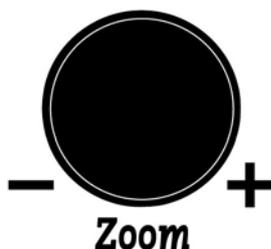

Use of the built-in object lighting


The double LED-lighting is built in to the camera arm for use in poorly-lit environments. The lighting can be switched on and off by pushing the zoom button in the following order: “step 1/ step 2/ Off”.

5. Operation of the Sentry+ [HD] control panel


Switch On / Off

Press the On / Off switch, located at the top right Corner of the control panel to switch on the Sentry+ [HD]. The On / Off indicator light will illuminate when the control panel is switched on. The monitor needs to be switched on separately.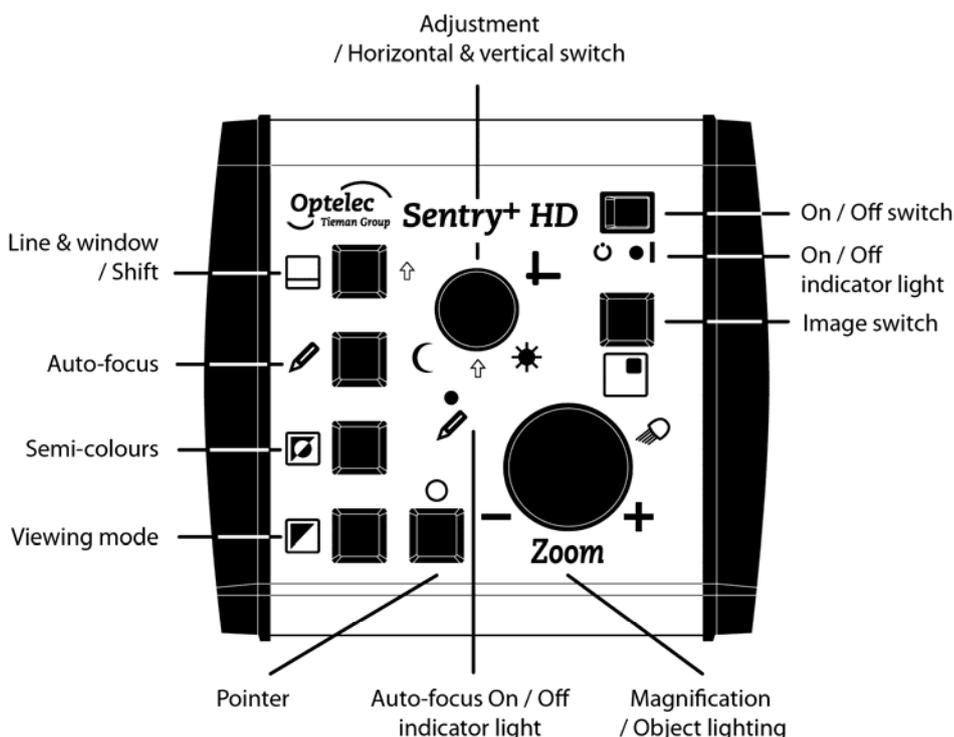

Setting the magnification

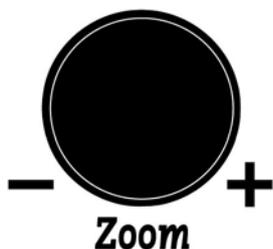Turning the Zoom control allows you to select the desired magnification level. The Sentry+ [HD] magnifies text and images from 1 to 70 times the original size on a 22-inch screen. Turning the circular control anti-clockwise will decrease the magnification. Similarly, turning the circular control clockwise will increase the magnification.

Wide screen mode – only at Sentry+

By pressing the Shift key along with the overview button you can switch between three views: Widescreen, flexible widescreen and without widescreen. Widescreen is the factory setting.

The overview mode

By pressing the overview key you can activate a crosshair target on the visual screen, which shows you the centre of the image of the camera. By pressing this key twice the crosshair target will disappear.

Viewing modes

Pressing this button will cycle you through three viewing modes:

1- Full Colour Photo mode

This mode will display text, pictures and objects in full colour.

2- Positive Reading mode

This mode enhances the foreground / background contrast. Pictures and text will be displayed in black text on a white background.

3- Negative Reading mode

This mode reverses the positive image. Pictures and text will be displayed as white text on a black background.


Semi-colours

You can activate this function in the Positive and Negative Reading Modes only. Pressing this button cycles you through a wide range of different foreground and background colour combinations. You may choose to display text in a pair of selectable colours when more contrast is needed or when alternate colour combinations provide easier reading.


Auto-focus On / Off

The Sentry⁺ [HD] features automatic focusing to produce a clear image at any time. When the auto-focus is not active the red LED indicator will illuminate. Press this button to switch to Focus Lock which stops the Sentry⁺ [HD] from continuously focusing. This is useful for tasks such as writing, painting or enjoying craftwork. Pressing this button will re-activate the auto-focus and the red LED indicator light will turn off again.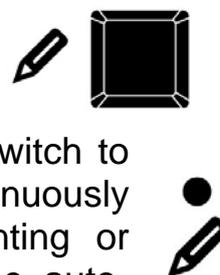


Contrast


The contrast changes with the adjustment button, which is presented on the right. Rotating this button counter clockwise decreases the contrast and rotating the button clockwise increases the contrast.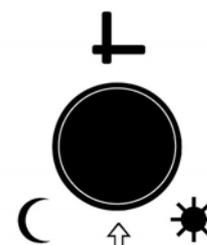

Brightness

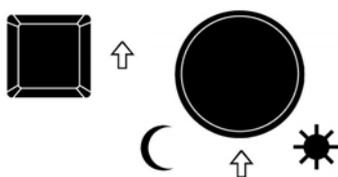
Turning the button whilst simultaneously pressing the adjacent Shift button allows you to adjust the brightness of the Sentry+ [HD] image. Turn the control anti-clockwise to decrease the brightness. Turn the circular control clockwise to increase the brightness.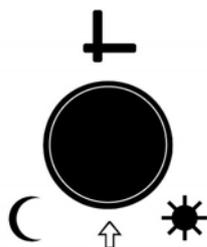

Activating line markers / window blinds

Pressing this button cycles you through line markers, window blinds and back to full screen. Horizontal line markers and window blinds are used to make reading easier.

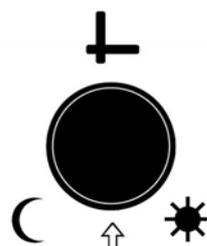Reading line and window shades adjustment

When line markers or window blinds are activated, turning this circular control will change their orientation.

Reading line colours

By pressing the Shift key along with the Semi-colours button you can change the reading line colours. The photo mode can be change separately. Black is the factory setting.

Key switch

This key enables additional functions. Pressing and holding this key while rotating the upper adjustment button allows you to change additional functions such as line distance, window shade distance and changes in brightness.

Picture switch

By pressing this button you can switch between the image from the Sentry+ [HD] camera and the PC image. If you have a footswitch available, you can control switching the image by pressing the footswitch.

Reset

Press this button when the Sentry+ [HD] is switched off. Switch the Sentry+ [HD] on while pressing this button and keep the button pressed for 2 seconds. The Sentry+ [HD] will now start with the original settings.

6. Sentry⁺ Specifications

- Camera : 18x optical zoom
- Viewing modes : Full colour photo mode
High contrast black-on-white
High contrast white-on-black
- Semi-colours : 8 foreground / background
colour combinations
- Magnification : 2 to 50 times on a 17" monitor
- Focus : Autofocus with focus lock and manual
focus
- Monitor output and input : DVI
- Screen frequency : 60 Hz
- Camera movements : 200° tilting and 360° swivel
- Light : LED
- Close-up lens : Removable
- Weight unit : 5.5 kg / 194 ounces
- Weight control panel : 0.85 kg / 30 ounces
- Weight power supply : 0.18 kg / 6.3 ounces
- Power supply input : 100-240 V 50-60 Hz AC
- Power supply output : 12 V / 2 Amp. DC

Operating conditions

- Humidity : < 75%, no condensation
- Temperature : 0 – 40 °C

Storage & transport conditions

- Humidity : < 75%, no condensation
- Temperature : -20 – 60 °C

WARNING: The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by the manufacturer of the Sentry⁺ as replacement parts for internal components, may result in increased emissions or decreased immunity of the Sentry⁺.

Optelec nor its partners do not take any responsibility when using the product and its accessories other than those specified in this manual.

For additional questions on the Sentry⁺ regarding service, installation or operation, contact your local distributor or nearest Optelec office (details at the end of this user manual).

7. Sentry⁺ HD Specifications

- Camera : 1080 HD 20x optical zoom
- Viewing modes : Full colour photo mode
High contrast black-on-white
High contrast white-on-black
- Semi-colours : 8 foreground / background
colour combinations
- Magnification : 2 to 50 times on a 17" monitor
- Focus : Autofocus with focus lock and manual
focus
- Monitor output and input : DVI
- Screen frequency : 60 Hz
- Camera movements : 200° tilting and 360° swivel
- Light : LED
- Close-up lens : Removable
- Weight unit : 5.5 kg / 194 ounces
- Weight control panel : 0.85 kg / 30 ounces
- Weight power supply : 0.18 kg / 6.3 ounces
- Power supply input : 100-240 V 50-60 Hz AC
- Power supply output : 12 V / 2 Amp. DC

Operating conditions

- Humidity : < 75%, no condensation
- Temperature : 0 – 40 °C

Storage & transport conditions

- Humidity : < 75%, no condensation
- Temperature : -20 – 60 °C

WARNING: The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by the manufacturer of the Sentry⁺ HD as replacement parts for internal components, may result in increased emissions or decreased immunity of the Sentry⁺ HD.

Optelec nor its partners do not take any responsibility when using the product and its accessories other than those specified in this manual.

For additional questions on the Sentry⁺ HD regarding service, installation or operation, contact your local distributor or nearest Optelec office (details at the end of this user manual).

8. Compliances

This product complies with the safety and EMC requirements of the directives:

EN60601-1: 2007-07

EN60601-1-2: 2007-12

Optelec

Breslau 4

2993 LT Barendrecht

The Netherlands

EU – Declaration of conformity

Barendrecht, the Netherlands

October 2008

The manufacturer declares that the Sentry⁺ and Sentry⁺ HD electronic magnifier complies to the safety guidelines and directives of the European Community.

This EU – declaration of conformity is based on tests according to the directives:

EN60601-1: 2007-07 and EN60601-1-2: 2007-12 by an independent Notified body.

Warning: The use of accessories, power supply units or cables which are not determined as original replacement equipment can cause higher emission or lower immunity of the Sentry⁺ and Sentry⁺ HD.

This device bears the CE approval mark
(UL/CSA/TÜV approved)

This electronic device is tested and shows compliance with the limits of the 60601-1 and the supplementary standards. These limits are determined to guarantee reasonable protection against harmful influences from other electronic devices. This electronic device uses electronic high frequency circuits and with this it produces legal electronic high frequency radiation. Improper use or false spare parts or accessories can influence other electronic devices in your household. Nevertheless there is no guarantee that influence in a particular matter excludes other electronic devices.

In case this device has a disturbing effect on other electronic receivers, at first you can try to solve the failure by the following procedures:

- Change the location of the receiving antenna of the interfered device.
- Enlarge the distance between the Sentry⁺ and your receiver.
- Use a different socket-outlet than the receiver.
- Ask your dealer or an electrician.

WARNING: You are cautioned that changes or modifications not expressly approved by the party responsible for compliance could void your authority to operate the equipment.

9. Warranty Conditions

Optelec warrants the Sentry⁺ [HD], effective from the date of delivery, to be free of any defects in material and workmanship.

Optelec reserves the right to repair or replace any purchased Sentry⁺ [HD] with a similar or improved product.

In no event shall Optelec or its suppliers be held liable for any indirect or consequential damages. The original user's remedies are limited to replacement of Sentry⁺ [HD] modules. This warranty is valid only when serviced in the country of original purchase, and with intact seals. For additional warranty claims or service during or after warranty period, please contact your Optelec distributor.

Optelec is not responsible for any uses of this device other than those described in this manual.

Optelec Headquarters**Optelec BV**

Breslau 4
2993 LT Barendrecht,
The Netherlands
T: +31 (0)88 6783 444
www.optelec.eu

Optelec Offices Worldwide**Optelec NV**

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Optelec UK

2 Millfield House, Woodshots Meadow, Croxley
Green Business Park,
Watford WD18 8YX
United Kingdom
T: +44 (1923) 23 13 13
www.optelec.co.uk

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Optelec GmbH
Optelec US Ltd
U.S.A.